Reuse and Cleaning Instructions for 10A Reusable Masks (non-medical use)

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1. The method and instructions here apply only to the 10A Reusable Masks (non-medical use)

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2. Please use your own commercially available $\geq 75\%$ ethanol spray for cleaning the masks as it is not included in the mask package.

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3. Please clean the mask before reuse. It is recommended that the 10A Reusable Masks be cleaned after 6-8 hours of use. If the mask is damaged, or exposed to hazardous/harmful pollutants, please dispose of it appropriately.

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4. To clean the reusable mask, please spray the outer surface (front) of the mask with $\geq 75\%$ ethanol spray:

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- a. Spray onto the outer surface (front) of the mask from 4 inches 3 times. Ensure the spray covers the whole outer surface (front) of the mask.

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- b. Avoid touching the outer layer (front) of the mask during the cleaning process.

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- c. After cleaning, air dry for about 1 hour. Avoid direct sunlight.

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- d. Only wear and reuse the mask when it is completely dry.

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5. The NANOGO™ filter element material used in the 10A Reusable Mask has been tested and the results show that it can be cleaned and reused up to 10 times, without any significant reduction in filtration efficiency while still maintaining its original protectability and functionality.

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6. Please pay close attention to the following:

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DO NOT share the mask with other people.

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DO NOT wash the mask using water or detergent.

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DO NOT use any brushes or other items to clean the mask.

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DO follow the instructions that come with the $\geq 75\%$ ethanol spray.

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When not in use, please store the mask in a suitable dry and clean container.

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DISCLAIMER:

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The information in this Information Sheet is believed to be correct as of the date issued.

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10A makes no warranties, expressed or implied, including, but not limited to, any implied warranty of merchantability or fitness for a particular purpose or course of performance or usage of trade.

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User is responsible for determining whether the 10A product is fit for a particular purpose and suitable for user's method of use or application.

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Given the variety of factors that can affect the use and application of a 10A product, some of which are uniquely within the user's knowledge and control, it is essential that the user evaluates the 10A product to determine whether it is fit for a particular purpose and suitable for user's method of use or application.

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10A provides information in electronic form as a service to its customers. Due to the remote possibility that electronic transfer may have resulted in errors, omissions or alterations in this information, 10A makes no representations as to its completeness or accuracy.

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